


STERILANCE[®]
m e d i c a l
Innovation for Safety



Sterilance™ Flex3 Depth Adjustable Safety Lancet


- 3 depths settings available in 1 device
- Improved ergonomic design
- Pre-loaded needle, contact activated

Instructions for use

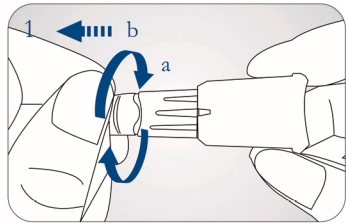


Twist off the cap first and then pull it straight out.

***It is important to completely twist the cap at 360° before removing it and DO NOT apply force to pull it out.**

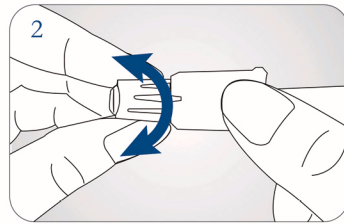


Rotate the adjustable tip (white part) to set the required depth.


- ||| Low depth(1.3mm)
- ||| Medium depth(1.8mm)
- ||| High depth(2.3mm)

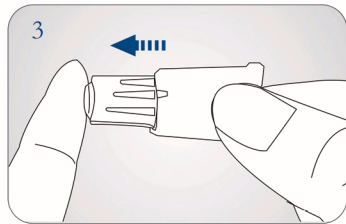

Place the lancet body firmly against the puncture site to activate the device.

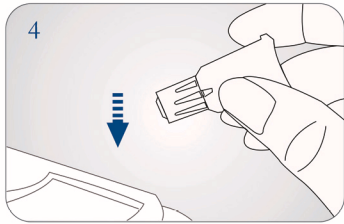

Dispose of the used lancet in a suitable sharps container.

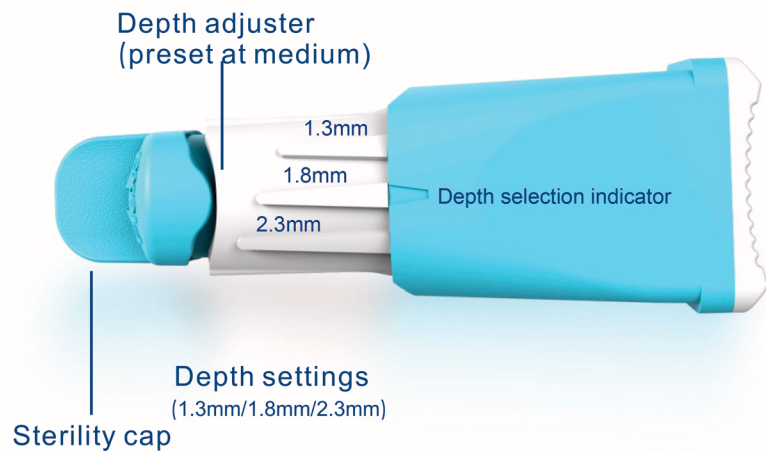

Ordering information

Reorder #	05-1521A	05-1523A	05-1526A	05-1528A	05-1530A
Specification	21G	23G	26G	28G	30G
Color	Gray	Blue	Yellow	Orange	Rose Red
Package	100pcs/box	10boxes/case or 1000pcs/case			

Easier twist-off cap design.

